

LUBRA SORB

STERILE AMORPHOUS HYDROGEL



STERILE AMORPHOUS HYDROGEL

LUBRASORB, is a Sterile Amorphous Hydrogel, helps to wound treatment, supporting natural autolytic debridement and provides a moist wound healing environment. Hydrogel creates a moist environment in wound surface. Thus, it helpful to removes the non-viable tissue, facilitating the debridement of dry necrotic tissue and lightly-exuding wounds and helps the wound heal. It provides removal of dry necrotic tissue and lightly-exuding tissue, especially in deep wounds.

LUBRASORB nourishes with moisture to the wound, and provides softening and moistening of necrotic tissues. Thus, it facilitates the removal of devitalized tissue. It absorbs exudates in lightly-exuding wounds. Thus, exudate, necrotic tissue and other non-viable tissue are removed from the wound in low exuding wounds with changing the wound dressing. Hydrogel, removal from wound, provides healing of deep and superficial wounds by supporting natural debridement. It is non-adhesive and does not damage when removed from the wound.

Indications: Hydrogel is suitable for treatment of arterial and venous ulcers, diabetic leg ulcers, pressure ulcer, superficial and partial burns (1. ve 2. degree), cuts and abrasions, post-operative surgical wounds, excoriated skin wounds, amputations, fungating ulcer and dry necrotic tissue or lightly-exuding wounds.

Content of hydrogel;


- Purified water
- Propylene glycol
- Carboxymethyl cellulose polymer

LUBRASORB is available in 6 mL of amorphous hydrogel in 6 mL syringes, 11 mL of amorphous hydrogel in 11 mL syringes, 15 mL of amorphous hydrogel in 20 mL syringes and 20 mL of amorphous hydrogel in 20 mL syringes. The syringe material is polypropylene. The syringe provides easy application to deep sores.

Note: LUBRASORB should be used with care in deep wounds based around the eyes and narrow opening. It should be used with care in body cavities where it is difficult to remove of the gel.



LUBRASORB

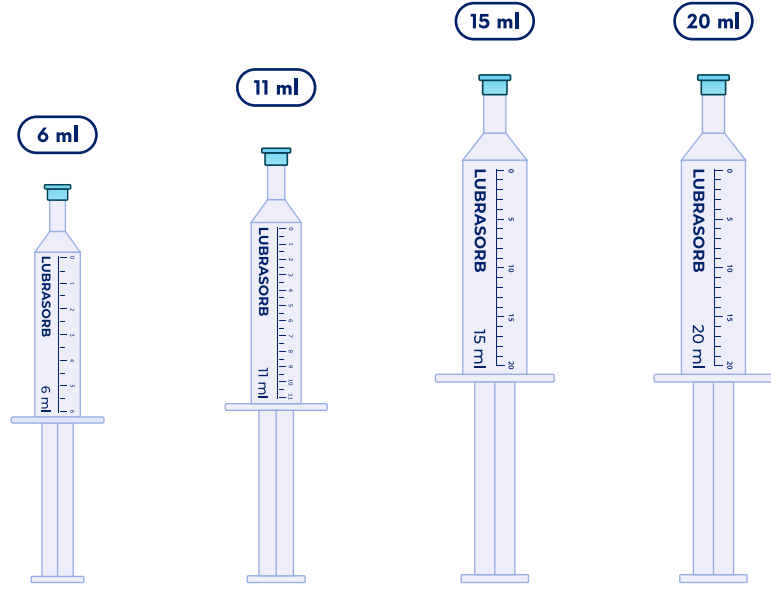

DIRECTION OF USE

- Cleanse the wound with 0.9 % sterile saline solution.
- Open the package containing product where indicated and remove syringe.
- Remove protective cap of product from syringe.
- Keeping to syringe tip from the wound, gently squeeze the syringe, and apply gel to whole wound, to depth of about 5 mm.
- The wound is closed by a suitable secondary dressing.

Cleaning of the Lubrasorb from the wound:

- LUBRASORB removes from the wound by rinsing with 0.9 % sterile saline solution.
- LUBRASORB may be left in place for up to 3 days as depending on the condition of the wound.
- Hydrogel must be changed, if wound exudate leakage occurs through a secondary dressing.



CE 2195

LATEX FREE

STERILE R


Istem
MEDİKAL TIBBİ CİHAZ VE SANAYİ LTD.ŞTİ.

LUBRA SORB

STERILE AMORPHOUS HYDROGEL

CATALOG NO.	DESCRIPTION	QTY/BOX
LSB-006	Sterile Amorphous Hydrogel 6ml	(25x6ml/Box)
LSB-011	Sterile Amorphous Hydrogel 11ml	(25x11ml/Box)
LSB-015	Sterile Amorphous Hydrogel 15ml	(20x15ml/Box)
LSB-020	Sterile Amorphous Hydrogel 20ml	(20x20ml/Box)